

Instructions For Use ZFN-IFU		
Rev. Date: July 23, 2009	Revision: 3	Page 1 of 1

P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 - Fax (435) 755-0015 - [www.scytek.com](http://www.scytek.com)

# Zinc Formalin Solution

**Description:** Zinc Formalin Solution is a tissue fixative for routine procedures and in various special stain procedures.

**Uses/Limitations:** For In-Vitro Diagnostic use only.  
Histological applications.  
Do not use past expiration date.  
Use caution when handling these reagents.

**Availability/Contents:**

<u>Item #</u>	<u>Volume</u>
ZFN125	125 ml
ZFN500	500 ml
ZFN999	1000 ml


**pH:** 4.0 (Adjust if desired).


**Storage:** Store at room temperature.

**Procedure (Standard):**


1. Trim specimen to smallest practical size.
2. Cover tissue with an amount of fluid equal to 20 times the specimen volume.
3. Fixation times range from 5 to 48 hours depending on specimen size.

**Precautions:** Poison. Do Not Take Internally.

Storage: 18° C  25° C

 ScyTek Laboratories, Inc.  
205 South 600 West  
Logan, UT 84321  
U.S.A.

CE 

 EmergoEurope (31)(0) 70 345-8570  
Molsnstraat 15  
2513 BH Hague, The Netherlands