

Vimentin

Availability/Contents:	<u>Item #</u>	<u>Volume</u>
	A20105	2 ml
	A00105	6 ml
	A00105.0025	25 ml

Description:


Species:	Rabbit
Immunogen:	Recombinant protein encoding human vimentin.
Mol. Weight:	57-60kDa
Clone:	SP20
Isotype:	IgG
Format:	This antibody has been preitered and quality controlled to work on formalin-fixed paraffin-embedded as well as acetone fixed cryostat tissue sections. No further titration is required.
Specificity:	This antibody selectively stains vimentin in human tissue sections. Vimentin is the main intermediate filament in mesenchymal cells and is therefore of value in the differential diagnosis of undifferentiated neoplasms.
Species Reactivity:	Human. Others not tested.
Positive Control:	Sarcoma and melanoma.
Cellular Localization:	Cytoplasmic.
Titer/Working Dilution:	No further dilution is required.
Microbiological State:	This product is not sterile.


Uses/Limitations: Not to be taken internally.
 For In Vitro Diagnostic Use.
 This product is intended for qualitative immunohistochemistry with normal and neoplastic formalin-fixed, paraffin-embedded tissue sections, to be viewed by light microscopy.
 Do not use if reagent becomes cloudy.
 Do not use past expiration date.

Storage and Stability: 2-8° Centigrade.
 Product is stable for 24 months from date of manufacture.
 If reagent is not stored as recommended, performance must be validated by the user.


Procedure:

1. **Primary Antibody Incubation Time:** We suggest an incubation period of 30 minutes at room temperature. However, depending upon the fixation conditions and the staining system employed, optimal incubation should be determined by the user.
2. **Visualization:** For maximum staining intensity we recommend the “UltraTek HRP Anti-Polyvalent Lab Pack” (ScyTek catalog# UHP125, see IFU for instructions) combined with the “DAB Chromogen/Substrate Bulk Pack (High Contrast)” (ScyTek catalog# ACV500, see IFU for instructions).

Storage: 2° C  8° C

 ScyTek Laboratories, Inc.
 205 South 600 West
 Logan, UT 84321
 U.S.A.

 EmergoEurope (31)(0) 70 345-8570
 Molsnstraat 15
 2513 BH Hague, The Netherlands

Instructions For Use

A00105-IFU-IVD

Rev. Date: Dec. 23, 2011

Revision: 1



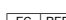
Page 2 of 2

P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 – Tel. (435) 755-9848 - Fax (435) 755-0015 - www.scytek.com

Precautions: Contains Sodium Azide as a preservative (0.09% w/v).
Do not pipette by mouth.
Avoid contact of reagents and specimens with skin and mucous membranes.
Avoid microbial contamination of reagents or increased nonspecific staining may occur.
This product contains no hazardous material at a reportable concentration according to U.S. 29 CFR 1910.1200, OSHA Hazardous Communication Standard and EC Directive 91/155/EC.

References:

Warranty: No products or “Instructions For Use (IFU)” are to be construed as a recommendation for use in violation of any patents. We make no representations, warranties or assurances as to the accuracy or completeness of information provided on our IFU or website. Our warranty is limited to the actual price paid for the product. ScyTek Laboratories, Inc. is not liable for any property damage, personal injury, time or effort or economic loss caused by our products. Immunohistochemistry is a complex technique involving both histological and immunological detection methods. Tissue processing and handling prior to immunostaining can cause inconsistent results. Variations in fixation and embedding or the inherent nature of the tissue specimen may cause variations in results. Endogenous peroxidase activity or pseudoperoxidase activity in erythrocytes and endogenous biotin may cause non-specific staining depending on detection system used.

Storage: 2° C  8° C ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.  EmergoEurope (31)(0) 70 345-8570
Molsnstraat 15
2513 BH Hague, The Netherlands