

Instructions For Use FMM-IFU		
Rev. Date: Jan. 3, 2008	Revision: 1	Page 1 of 1

P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 - Fax (435) 755-0015 - www.scytek.com

FluoreGuard Mounting Medium

Description: FluoreGuard Mounting Medium is a water-soluble medium designed for semi-permanent coverslipping of fluorescent slide preparations. FluoreGuard dries to a semi-rigid layer which eliminates tissue damage due to moving coverslips and facilitates long term storage.

Uses/Limitations: For In-Vitro Diagnostic use only.
Histological applications.
Do not use past expiration date.
Use caution when handling these reagents.

Availability:


<u>Item #</u>	<u>Volume</u>
FMM030	30 ml
FMM060	60 ml


Storage: Store at room temperature.

Procedure:

1. Apply several drops of FluoreGuard Mounting Medium and coverslip.
2. Allow FluoreGuard to dry for 30 minutes prior to viewing. Product will dry to a semi-solid form overnight.
3. FluoreGuard is an aqueous based medium and may be removed by soaking in PBS overnight.

Precautions: Avoid contact with skin and eyes. Wash after use. Observe all federal, state and local environmental regulations regarding disposal.

Storage: 18° C  25° C

 ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
435-755-9848
U.S.A.

CE Authorized Representative in Europe
 (Regulatory affairs only)
EmergoEurope (31)(0) 70 345-8570
Molsnstraat 15
2513 BH Hague, The Netherlands