

Light Green Solution (0.1%)

Description: Light Green Solution (0.1%) is intended for use in histological applications as a counterstain for Grocott's Silver Stain in addition to other procedures. When used correctly, various shades of green can be obtained to aid in visualization of tissue components.

Cytoplasm:	Light Green
Nuclei:	Light Green/Blue

Uses/Limitations: Not to be taken internally.
 For In-Vitro Diagnostic use only.
 Histological applications.
 Do not use past expiration date.
 Use caution when handling reagents.
 Non-Sterile.

Availability:	<u>Item #</u>	<u>Volume</u>
	LGD125	125 ml
	LGD500	500 ml
	LGD999	1000 ml


Storage: Store at 18-25°C.

Precautions: Avoid contact with skin and eyes.
 Harmful if swallowed.
 Follow all Federal, State, and local regulations regarding disposal.

Procedure:

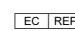
1. Deparaffinize sections if necessary and hydrate to distilled water.
2. Rinse slide in distilled water.
3. Stain slide in Light Green Solution (0.1%) for 1-2 minutes.
4. Rinse 1 time in Distilled/Deionized water.
5. Dehydrate quickly through graded alcohols.
6. Clear, and mount in synthetic resin.

Storage: 18° C  25° C



ScyTek Laboratories, Inc.
 205 South 600 West
 Logan, UT 84321
 U.S.A.

CE 

 EmergoEurope (31)(0) 70 345-8570
 Molsnstraat 15
 2513 BH Hague, The Netherlands