

Instructions For Use WSC-IFU		
Rev. Date: Oct. 18, 2010	Revision: 1	Page 1 of 1

P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 – Tel. (435) 755-9848 - Fax (435) 755-0015 - www.scytek.com

Wash Solution Concentrate (50x)

Description: Wash Solution Concentrate (50x) is designed to provide optimal rinsing of routine or special stains between steps. In addition, this reagent can be used as a dilution buffer for most aqueous stains. This product is compatible with hand staining procedures and most automated systems.

Appearance: Transparent liquid, slightly yellow. Discard if reagent becomes cloudy.

Uses: Routine and Special Stain procedures.
Stain diluent.

Limitations: Do not use past expiration date.

Availability:


<u>REF</u>	<u>Volume</u>
WSC-1	4 x 200 ml
WSC999	1000 ml
WSC20000	20 liters


Storage: Store at 18-25°C. Product is stable for 24 months from the date of manufacture.

Procedure:

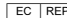
1. Add 200ml of Wash Solution Concentrate (50x) to 9.8 liters of DI/Distilled water and mix thoroughly at low speed to avoid “foaming” the reagent.
2. Mix fresh daily for optimal results.

Precautions: Do not pipette by mouth.
Avoid contact with skin and eyes.
Observe all federal, state and local environmental regulations regarding disposal.

Storage: 18° C  25° C

 ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.

CE 

 EmergoEurope (31)(0) 70 345-8570
Molsnstraat 15
2513 BH Hague, The Netherlands