



Instructions For Use

HCS1001-IFU

205 South 600 West Logan, Utah 84323, U.S.A. – Tel. (800) 729-8350 – Tel. (435) 755-9848 – Fax (435) 755-0015 – www.scytek.com Rev. 1, 5/6/2024

Acid Fast Bacteria Control Slides

Description

Acid Fast Bacteria Control Slides contain formalin fixed paraffin embedded sections of Human or Animal tissue cut at 4 micron thickness. Slides are known to stain positive for acid-fast mycobacteria.

Storage

Store slides at 2-25°C

Suggested Stain Kit (not provided)

FAB-1

See below procedure

Acid Fast Bacteria (AFB Stain Kit)

Description and Principle

The Acid Fast Bacteria (AFB) Stain Kit is intended for use in the histological visualization of bacteria characterized as "acid-fast". AFB is especially useful detecting mycobacteria such as *Mycobacterium tuberculosis*.

Acid-fast organisms take up Carbol Fuchsin and resist decolorization due to high lipid content of the cell wall. Carbol Fuchsin is removed from organisms without a lipid cell wall by a dilute acid.

Expected Results

Acid Fast Organisms:	Bright Red
Background:	Light Green

Kit Contents

1. Carbol Fuchsin Solution
2. Acid Alcohol Solution (0.5%)
3. Light Green Solution

Storage

- 18-25°C
- 18-25°C
- 18-25°C.

Suggested Controls (not provided)

Tissue or cell smear containing acid-fast organisms

Uses/Limitations

For In-Vitro Diagnostic use only.

Do not use if reagents become cloudy or precipitate

Do not use past expiration date.

Use caution when handling reagents.

Non-Sterile

Intended for FFPE sections cut at 5-10µm.

This procedure has not been optimized for frozen sections.

Frozen sections may require protocol modification.


Storage

Store kit and all components at room temperature (18-25°C).

Safety and Precautions

Please see current Safety Data Sheets (SDS) for this product and components GHS classification, pictograms, and full hazard/precautionary statements.

Note: Carbol Fuchsin Solution should be filtered when a thick sheen develops on top of solution. To avoid possible contamination, gloves should be worn when performing this procedure. Do not use tap water prior to application of Carbol Fuchsin Solution as it is reported that Acid Fast Bacteria can be found in some systems. Therefore use of distilled water is recommended whenever possible

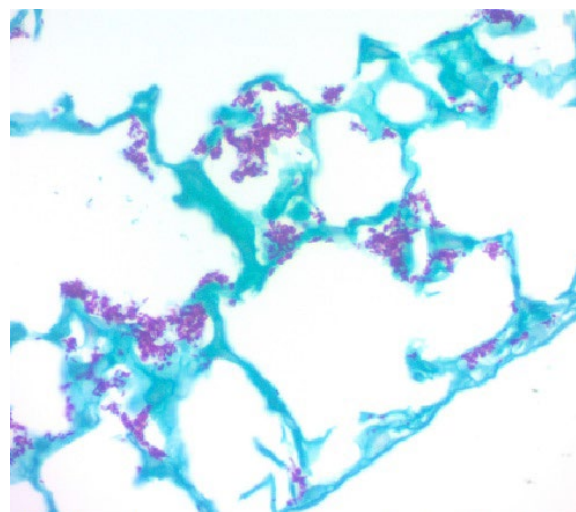

Mycobacterium gordonae artificially introduced in Mouse Lung demonstrated by Acid Fast Bacteria Stain.

Procedure

1. Deparaffinize sections if necessary and hydrate to distilled water.
2. Incubate slide in Carbol Fuchsin Solution for 15 minutes.
3. Rinse for 2 minutes in running tap water followed by 2 changes of distilled water.
4. Decolorize in Acid Alcohol Solution (0.5%) until sections are a pale pink color.
5. Dip slide twice in distilled water to rinse.
6. Counterstain in Light Green Solution for 1-2 minutes.
7. Rinse in several changes of distilled water.
8. Dehydrate quickly through graded alcohols ending with 2 changes in absolute alcohol.
9. Clear, and mount in synthetic resin.

References

1. Sheenan, D.C., Hrapchak, B.B. Theory and Practice of Histotechnology, 2nd Edition. Battelle Press, Columbus, OH.
2. Carson, F.L., 1996, Histotechnology; A Self-Instructional Text, 2nd Edition. ASCP Press, Chicago, IL.



ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
435-755-9848
U.S.A.