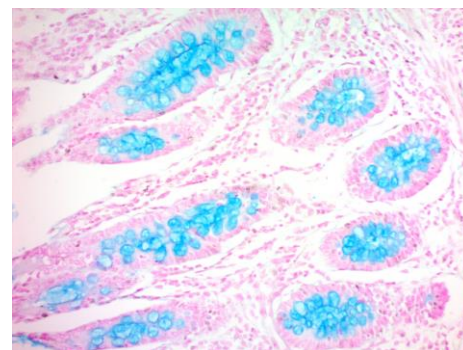


Alcian Blue (pH 1.0) Stain Kit

Description: The Alcian Blue (pH 1.0) Stain Kit is intended for use in the histological visualization of strongly sulfated mucosubstances. Alcian blue staining at pH 1.0 differs from staining at pH 2.5, in that carboxylated mucins are protonated and only sulfated mucins are still ionized and therefore stained.

Strongly Sulfated Mucosubstances: Blue
 Nuclei: Red
 Background: Pink



Uses/Limitations: Not to be taken internally.
 For In-Vitro Diagnostic use only.
 Histological applications.
 Do not use past expiration date.
 Use caution when handling reagents.
 Non-Sterile.

Control Tissue: Tissue known to be positive for sulfated mucins. e.g. deep mucosa of colon

Availability/Contents:

<u>Item #</u>	<u>Kit Contents</u>	<u>Volume</u>	<u>Storage</u>
ANA030	Alcian Blue Solution (pH 1.0)	30 ml	18-25°C
HQA030	Hydrochloric Acid Solution (1N)	30 ml x 2	18-25°C
NFS030	Nuclear Fast Red (Enhanced Stability)	30 ml	18-25°C

Precautions: Avoid contact with skin and eyes.
 Harmful if swallowed.
 Follow all Federal, State, and local regulations regarding disposal.
 Use in chemical fume hood whenever possible.

Procedure notes:

Maintaining proper pH is critical to preventing false-positive mucin staining. 'Working Rinse Solution' should fall within pH 1.0 ± 0.15 and is used both before and after the Alcian Blue to control pH. Rinsing with deionized water or any other rinse may affect pH and cause non-specific staining. We recommend running at least one slide without the counterstain to compare intensity with slides that have been counterstained. Below procedure is written for slides laying horizontally that are stained by applying a small amount of solution.

Procedure

1. Deparaffinize sections if necessary and hydrate to distilled water.
2. Make up sufficient amount of 'Working Rinse Solution' by mixing the following:


1 part of Hydrochloric Acid Solution (1N)

9 parts Deionized water

Notes: -An example mixture would be 10mls Hydrochloric Acid Solution + 90mls Deionized Water

-We suggest making at least 10mls per slide. A smaller amount is required for step 3 and a larger amount used in step 5 for rinsing.

Storage: 18° C  25° C

 ScyTek Laboratories, Inc.
 205 South 600 West
 Logan, UT 84321
 U.S.A.

CE 

EC REP
 Emergo Europe
 Prinsessegracht 20
 2514 AP The Hague, The Netherlands

Instructions For Use AFT-2-IFU

Rev. Date: Aug. 31, 2020

Revision: 4

Page 2 of 2


P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 – Tel. (435) 755-9848 - Fax (435) 755-0015 - www.scytek.com

3. Apply a small amount of 'Working Rinse Solution' to tissue for 30 seconds to adjust pH in preparation for staining. Save remaining 'Working Rinse Solution' for step 5.
4. Drain slide and without rinsing, stain tissue section with Alcian Blue Solution (pH 1.0) solution for 30 minutes
5. Quickly and thoroughly rinse excess stain off slide using remaining 'Working Rinse Solution'.
6. Carefully blot and allow slide to air dry.
7. If preferred, counter stain in Nuclear Fast Red (Enhanced Stability) for 1-2 minutes with occasional agitation. Rinse very briefly in deionized water and allow to completely air dry again.
8. Clear, and mount in synthetic resin.

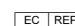
References:

1. Lev, R., Spicer, S.S. 1964, Specific Staining of sulfate groups with alcian blue at low pH. Journal of Histochemistry and Cytochemistry. Volume 12, Page 309.
2. Pennock, C.A., Burns, J., Massarella, G. 1968, Histochemical investigation of acid mucosubstances in secondary amyloidosis.
3. Lillie, R.D. 1977, H.J. Conn's Biological Stains, 9th Edition. Williams & Wilkins, Baltimore. Pages 452-455.
4. Sheenan, D.C., Hrapchak, B.B. Theory and Practice of Histotechnology, 2nd Edition. Battelle Press, Columbus, OH. Pages 172-173.
5. Churukian, C.J., 1989, Manual of Special Stains Laboratory, 4th Edition. University of Rochester, Rochester, New York. Pages 55-56.
6. Carson, F.L., 1996, Histotechnology; A Self-Instructional Text, 2nd Edition. ASCP Press, Chicago, IL. Pages 117-121.

Storage: 18° C  25° C

 ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.


Emergo Europe
Prinsessegracht 20
2514 AP The Hague, The Netherlands