


Reticulum Stain Kit

Description: The Reticulum Stain Kit is intended for use in histological demonstration of reticular fibers. The main function of reticular fibers is to provide support. They are normally found throughout the body, particularly in liver, lymph node, spleen and kidney. Ammoniacal silver stains are the most commonly used methods for demonstration of reticular fibers.

Reticulum: Black
Background: Green

Control Tissue: Liver
Kidney
Lymph Node
Spleen


Uses/Limitations: Not to be taken internally.
For In-Vitro Diagnostic use only.
Histological applications.
Do not use if reagents become cloudy.
Do not use past expiration date.
Use caution when handling reagents.
Non-Sterile

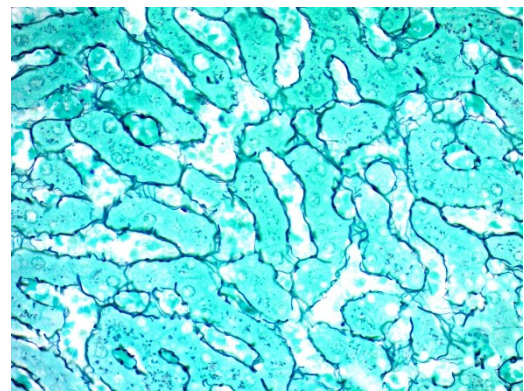

Kit Contents:


<u>Item #</u>	<u>Kit Contents</u>	<u>Volume</u>	<u>Storage</u>
PPE015	Potassium Permanganate Solution (1%)	15 ml	18-25° C.
PMS015	Potassium Metabisulfite Solution (3%)	15 ml	18-25° C.
FAT015	Ferric Ammonium Sulfate Solution (3%)	15 ml	18-25° C.
FRL015	Formalin Solution (20%)	15 ml	18-25° C.
GCS015	Gold Chloride Solution (0.1%)	15 ml	2-8° C.
STB015	Sodium Thiosulfate Solution (5%)	15 ml	18-25° C.
LGA015	Light Green Solution	15 ml	18-25° C.
SHC010	Sodium Hydroxide Solution (3%)	10 ml x 2 vials	18-25° C.
SNX010	Silver Nitrate Solution (10%)	10 ml x 2 vials	2-8° C.

Precautions: Keep away from open flame.
Avoid contact with skin and eyes.
Harmful if swallowed.
Follow all Federal, State, and local regulations regarding disposal.
Use in chemical fume hood whenever possible.
Wear protective clothing.

Storage: 2° C  25° C

**Mixed Storage Conditions.
Separate Contents.**

Doc: IFU-TemplateMixedStorageev2



ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.

CE IVD

EC REP
Emergo Europe
Prinsessegracht 20
2514 AP The Hague, The Netherlands


Preparation of Reagents Prior to Beginning:

1. Prepare working Ammoniacal Silver Solution using chemically cleaned glassware in a chemical fume hood as follows: To 17ml of Distilled Water add one (10ml) vial of Silver Nitrate Solution (10%) and mix completely. Add 1ml of Concentrated Ammonium hydroxide (25-30%) (**Not Included in Kit**) while continuously mixing. The mixture will initially turn brown and then become clear. Add one (10ml) vial of Sodium Hydroxide Solution (3%) and mix completely. If the solution does not remain colorless, add Concentrated Ammonium hydroxide (25-30%) drop by drop until no precipitate remains. Add Distilled Water to a total volume of 60ml and mix completely. Solution is now ready for use.

Note: Use extreme care in preparation and use of Ammoniacal Silver Solution. Store Ammoniacal Silver Solution in a refrigerator to avoid the formation of explosive compounds. If Ammoniacal Silver Solution is exposed to sunlight, it will explode. Dispose of waste observing all local, state and federal laws.

Procedure (Standard):

1. Deparaffinize sections if necessary and hydrate to distilled water.
2. Place slide in Potassium Permanganate Solution (1%) for 5-10 minutes.
3. Rinse in 3 changes of distilled water.
4. Differentiate in Potassium Metabisulfite Solution (3%) until section is transparent.
5. Rinse in 3 changes of distilled water.
6. Apply Ferric Ammonium Sulfate Solution (3%) for 10 minutes.
7. Rinse in 2 quick changes of distilled water.
8. Apply working Ammoniacal Silver Solution for 2-3 minutes.
9. Rinse in 3 changes of distilled water.
10. Place slide in Formalin Solution (20%) for 1 minute.
11. Rinse in 3 changes of distilled water.
12. Apply Gold Chloride Solution (0.1%) for 3-5 minutes.
13. Rinse in 2 changes of distilled water.
14. Apply Sodium Thiosulfate Solution (5%) for 1-2 minutes to remove unreduced silver.
15. Rinse in tap water for 2 minutes.
16. Counterstain using Light Green Solution for 2-5 minutes.
17. Rinse in tap water followed by distilled water.
18. Dehydrate through 3 changes of Absolute Alcohol.
19. Clear, and mount in synthetic resin.

Storage: 2° C  25° C

**Mixed Storage Conditions.
Separate Contents.**

Doc: IFU-TemplateMixedStorageev2




ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.



Emergo Europe
Prinsessegracht 20
2514 AP The Hague, The Netherlands

References:

1. Carson, FL., Histotechnology: A Self Instructional Text, ASCP Press, Chicago, IL. Pages 150-155, 1990.
2. Sheenan, D.C., Hrapchak, B.B. Theory and Practice of Histotechnology, 2nd Edition. CV Mosby, St. Louis, MO. Pages 181-182, 1980.
3. Churukian, C.J., Prolonging the Shelf-life of Solutions Containing Silver Nitrate. Histologic, Volume 10, Page 147, 1980.
4. Churukian, C.J., Modified Gomori's method for staining reticulum and collagen. Histologic, Volume 2, Page 23, 1972.
5. Wellington, EF., The Explosive properties of ammoniacal-silver solutions. Journal of Medical Lab Technology, Volume 22, Pages 220-223, 1965.
6. Gomori, G., A Modification of the Silver Impregnation Method of Staining Reticular Fibers. American Journal of Clinical Pathology, Volume 21, Pages 897-899, 1951.
7. Gomori, G., Silver Impregnation of Reticulum in Paraffin Sections. American Journal of Clinical Pathology, Volume 13, Pages 993-1002, 1937.

Storage: 2° C  25° C**Mixed Storage Conditions.
Separate Contents.**

Doc: IFU-TemplateMixedStorageev2

ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.Emergo Europe
Prinsessegracht 20
2514 AP The Hague, The Netherlands