

Eosin Y Solution (Modified Alcoholic)


Description: Eosin Y Solution (Modified Alcoholic) is intended for use in histology and cytology applications. This newly formulated Eosin provides the benefits of a traditional alcoholic formula with significant improvements in usability. Advantages include lower evaporation rate, better color patterns, reduced tendency to spill over container, hands, and countertops, and improved surface tension to remain on tissue section.

Cytoplasm:	Light Pink
Collagen:	Pink
Muscle:	Pink/Rose
Erythrocytes:	Pink/Red

Uses/Limitations: Not to be taken internally.
For In-Vitro Diagnostic use only.
Histological applications.
Do not use if reagents become cloudy.
Do not use past expiration date.
Use caution when handling reagents.
Non-Sterile.

Control Tissue: Any well fixed paraffin embedded or frozen tissue section.
Cell smear.


Ordering Information and Current Pricing at www.scytek.com

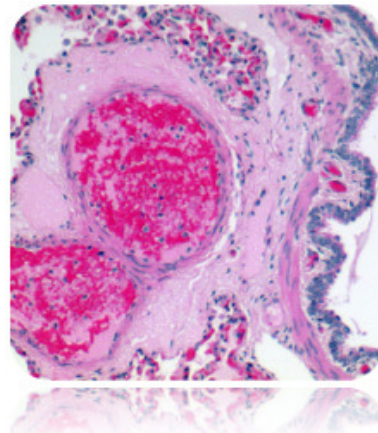


Availability:	<u>Item #</u>	<u>Volume</u>
	EYB500	500 ml
	EYB999	1000 ml
	EYB3800	1 Gallon

Precautions: Avoid contact with skin and eyes.
Flammable.
Harmful if swallowed.
Follow all Federal, State, and local regulations regarding disposal.

Storage: 18° C  25° C

 ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.

 EmergoEurope (31)(0) 70 345-8570
Molsnstraat 15
2513 BH Hague, The Netherlands

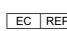Instructions For Use
EYB-IFU

Rev. Date: Feb, 21, 2012

Revision: 1

Page 2 of 2

P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 – Tel. (435) 755-9848 - Fax (435) 755-0015 - www.scytek.com

Procedure:

1. Deparaffinize sections if necessary and hydrate to distilled water.
2. Dip slide in absolute alcohol and blot excess off.
3. Apply adequate Eosin Y Solution (Modified Alcoholic) to completely cover tissue section to excess and incubate for 2-3 minutes.
4. Rinse slide using absolute alcohol.
5. Dehydrate slide in 3 changes of absolute alcohol.
6. Clear slide and mount in synthetic resin.

References:

Storage: 18° C



25° C



ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.



EC REP EmergoEurope (31)(0) 70 345-8570
Molsnstraat 15
2513 BH Hague, The Netherlands