

Trichrome Stain Kit (Modified Gomori's)

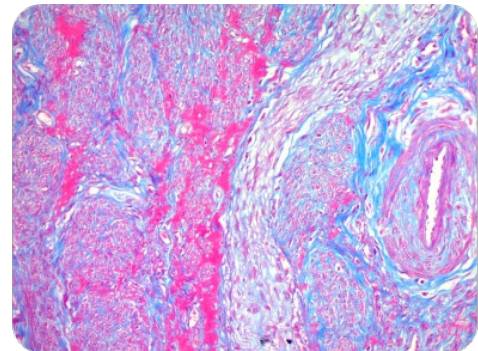
Description:

The Trichrome Stain Kit (Modified Gomori's) is intended for use in the histological visualization of collagenous connective tissue fibers in tissue sections. The Gomori Trichrome is a more simplified procedure than other more traditional trichromes. Additionally, this procedure is useful in the assessment of the degree of fibrosis in liver biopsies.

Collagen:	Blue
Muscle Fibers:	Red
Nuclei:	Dark Red to Black/Blue

Uses/Limitations:

Not to be taken internally.
For In-Vitro Diagnostic use only.
Histological applications.
Do not use if reagents become cloudy.
Do not use past expiration date.
Use caution when handling reagents.
Non-Sterile.


Control Tissue:

Lung
Uterus
Small Intestine
Appendix

Availability/Contents:

<u>Item #</u>	<u>Kit Contents</u>	<u>-1 Kit</u>	<u>-2 Kit</u>	<u>-500 Kit</u>	<u>Storage</u>
BNF125	Bouin's Fluid	125 ml	2 x 30 ml	500 ml	18-25°C
HWI-A-125	Hematoxylin, Weigert's Iron (Part A)	125 ml	30 ml	500 ml	18-25°C
HWI-B-125	Hematoxylin, Weigert's Iron (Part B)	125 ml	30 ml	500 ml	18-25°C
TGB125	Trichrome Stain Solution (Blue)	125 ml	30 ml	500 ml	18-25°C
AAD250	Acetic Acid Solution (0.5%)	250 ml	60 ml	500 ml	18-25°C
AAL250	Acid Alcohol Solution (0.5%)	250 ml	60 ml	500 ml	18-25°C

Ordering information regarding individual components on back page!

Precautions:

Keep away from open flame.
Avoid contact with skin and eyes.
Harmful if swallowed.
Follow all Federal, State, and local regulations regarding disposal.
Use in chemical fume hood whenever possible.

Storage: 18° C



25° C



ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
435-755-9848
U.S.A.



EC REP

Emergo Europe
Prinsessegracht 20
2514 AP The Hague, The Netherlands

Procedure:

1. Deparaffinize sections if necessary and hydrate to distilled water.
2. Preheat Bouin's Fluid in a water bath to 62° - 64° centigrade in a fume hood or very well ventilated area.
3. Place slides in preheated Bouin's Fluid for 60 minutes followed by a 10 minute cooling period.
4. Rinse slide in tap water until section is completely clear.
5. Rinse 1 time in distilled water.

Note: Use mixed Hematoxylin, Weigert's Iron immediately and dispose of directly thereafter.

6. Mix an equal volume of Hematoxylin, Weigert's Iron (Parts A and B) and apply to tissue section for 5 minutes.
7. Rinse slide in tap water for 2 minutes.
8. Apply Acid Alcohol Solution (0.5%) to slide for 5 seconds to adjust pH and remove excess stain.
9. Rinse slide in tap water for 1 minute.
10. Rinse slide 1 time in distilled water.
11. Apply Trichrome Stain (Blue) to tissue section and incubate for 15 minutes.
12. Rinse slide quickly in distilled water.
13. Apply Acetic Acid Solution (0.5%) to tissue section for 10 seconds.
14. Rinse slide using absolute alcohol.
15. Dehydrate in 2 changes of absolute alcohol, clear, and mount in synthetic resin.

References:

1. Gomori, G.L., A rapid one-step trichrome. Amer J of Clinical Pathology. Vol. 20: page 661, 1950.
2. Manual of Histologic Staining Methods of AFIP. 3rd Edition, McGraw-Hill Publishing. Page 93, 1968.
3. Elbadawi, A., Hexachrome modification of Movat's stain. Stain Technology, Vol. 51: pages 249-253, 1976.
4. Carson, F.L., Histotechnology; A Self-Instructional Text, 2nd Edition. ASCP Press, Chicago, IL. Pages 117-121, 1996.
5. Tretheway, D., et al. Should Trichrome Stain Be Used on All Post-Liver Transplant Biopsies with Hepatitis C Virus Infection To Estimate the Fibrosis Score? Liver Transplantation, May; 14(5): pages 695-700, 2008.

Storage: 18° C



25° C



ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
435-755-9848
U.S.A.





EC REP

Emergo Europe
Prinsessegracht 20
2514 AP The Hague, The Netherlands

Bulk Reagent Ordering Information and Current Pricing at www.scytek.com


Description:	Catalog #	Volume
Bouin's Fluid	BNF125	125 ml
	BNF500	500 ml
	BNF999	1000 ml
Hematoxylin, Weigert's Iron Kit (Two Components)	HWI-1	250 ml
	HWI-2	1000 ml
	HWI-3	2000 ml
Trichrome Stain Solution (Blue)	TGB125	125 ml
	TGB500	500 ml
	TGB999	1000 ml
Acetic Acid Solution (0.5%)	AAD250	250 ml
	AAD500	500 ml
	AAD999	1000 ml
Acid Alcohol Solution (0.5%)	AAL250	250 ml
	AAL500	500 ml
	AAL999	1000 ml

 Storage: 18° C  25° C



ScyTek Laboratories, Inc.
 205 South 600 West
 Logan, UT 84321
 435-755-9848
 U.S.A.




 Emergo Europe
 Prinsessegracht 20
 2514 AP The Hague, The Netherlands